# Bio Desis

ESP

## IMPLANTE DE MATRÍZ ÓSEA BOVINA

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## BOVINE BONE MATRIX IMPLANT

Particles ranging from 1.68 to 4mm

10cc single-dose vial
Batch No.: see product
Expiry date: see product

### COMPOSITION:

Bio Desis is a bovine bone mineral.The highly purified mineral osteo-conductive structure is made from natural bone through a multistage purification process that strictly complies with the safety standards. Due to its natural origin, Bio Desis is structurally and chemically comparable to mineralized human bone (natural nanocrystalline apatite).

### PROPERTIES/EFFECTS:

**The product is provided sterile. Do not sterilize again. Non-Toxic.**

The Bio Desis inorganic bone matrix has a macroporous and microporous structure similar to that of the human cancellous bone.The natural composition and the large volume of interconnected pores favor the formation and growth of new bone at the implantation site. Over time, Bio Desis undergoes partial remodeling as a result of the action of osteoclasts and osteoblasts (physiological remodeling). Due to its properties, Bio Desis is a viable alternative to autogenous bone in suitable defects.

### INDICATIONS:

Bio Desis is indicated as a bone substitute and/or bone filler of osteo-conductive and osteoinductive action.

Its indications include:

- Primary and revision joint replacement
- Spinal surgeries
- Traumatology in closed fractures and osteotomies
- Osteosynthesis
- Post-tumor resection filling
- Dental surgery
- Cranioplasties

### LIMITATIONS OF USE/PRECAUTIONS:

#### Contraindications

Bio Desis must not be used in infected areas, and in cases of structural or pathological conditions of the bone which are considered to prevent good bone growth.

### PRECAUTIONS

**Material for single use. Its reuse is prohibited. Do not use if package is not complete. Use diluted in patient’s blood or saline solution. Sterile. Its resterilization is prohibited.**

Material for use in its entirety on a single receiver.

Bio Desis must only be used by authorized professionals trained in the relevant surgical techniques such as dentists, surgeons or orthopedists. Bio Desis must be used with special caution in patients with:

- Acute or chronic infection at the surgical site. (E.g., osteomyelitis)
- Uncontrolled metabolic diseases such as diabetes, osteomalacia, severe renal or liver disease, thyroid disorder
- Auto immune diseases
- Radiotherapy
- Prolonged corticosteroid therapy
- High consumption of tobacco or alcohol
- Malnutrition
- Suspected foreign body sensitivity or allergies. (Studies must be done to determine what material to implant)
- Psychological condition that prevents treatment

- Suspected foreign body sensitivity or allergies. (Studies must be done to determine what material to implant)

- Suspected foreign body sensitivity or allergies. (Studies must be done to determine what material to implant)