OstiumMAX Cover OstiumMAX Cover S

ESP

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RESORBABLE COLLAGEN MEMBRANE

PRESENTATION OF THE PRODUCT

The product comes as a sterile porcine collagen membrane of various sizes: 1x1.5 cm, 1.5x2 cm., 2x3 cm.

COMPOSITION: PORCINE COLLAGEN 100%

Ostium Max Cover, OstiumMAX Cover S is an irradiated collagen membrane of porcine origin. The highly purified protein matrix is made from natural porcine collagen through a multi-stage purification process that strictly complies with safety standards.

PROPERTIES/EFFECTS:

Ostium Max Cover, OstiumMAX Cover S is a sterilized soft tissue (porcine collagen) that performs a proper long-term barrier function. Thanks to a unique manufacturing process, the superior properties of native porcine collagen are preserved without altering the characteristics of this natural tissue. Its use favors and accelerates bone regeneration, as it prevents invasion of soft tissue. Ostium Max Cover, OstiumMAX Cover S is completely resorbable so no complementary surgeries are needed.

INSTRUCTIONS FOR USE:

For the use of Ostium Max Cover, OstiumMAX Cover S, the general principles of sterile handling and patient medication must be followed.

- If necessary, the bone defect is surgically exposed and the usual surgical procedures (curettage, etc.) are performed.
- The defect is filled with a spacer, such as autogenous bone or a bone substitute (Ostium Max) whenever possible.
- The membrane is trimmed with scissors to the required size. It may be useful to use a template. The membrane should overlap the defect by at least 2 mm to allow full bone contact and to prevent the growth of connective tissue on the sides of the material.
- Hydrate with sterile saline, for approximately 30 minutes.
- The membrane is applied over the defect and held in place with moderate pressure. The time required for the application of pressure will depend on the degree of bleeding.
- The complete penetration of blood and exudates into the membrane enables the perfect adaptation and adhesion of the membrane to the defect and facilitates the formation of a blood clot under the membrane.
- Membrane adhesion is possible thanks to its high tensile strength. Adhesion may be indicated to prevent the movement of the membrane due to shear loads or mobilization.
- If possible, the soft tissue flap is sutured to the membrane securely and without tension (e.g., single sutures and deep mattress suture).
- In general, if possible, the wound should be closed completely to avoid a potential accelerated absorption of the exposed membrane.

Special Instructions for Use in Periodontology

- It is recommended that a hygiene phase be carried out prior to surgery, which will include the provision of appropriate instructions to the patient, since a basic requirement for successful treatment is the control of bacterial infections such as meticulous

oral hygiene. A postoperative maintenance phase can ensure satisfactory treatment in the long term.

- To avoid the formation of long junctional epithelium, Ostium Max Cover, OstiumMAX Cover S must be firmly attached to the tooth (if necessary, with additional attachment with suture material).

LIMITATIONS OF USE/PRECAUTIONS

CONTRAINDICATIONS

- Ostium Max Cover, OstiumMAX Cover S is contraindicated in:
- patients suffering from acute infections in the oral cavity.
- patients with known allergy to collagen.

PRECAUTIONS

Ostium Max Cover, OstiumMAX Cover S should only be used by licensed professionals with training in this field, such as dentists and surgeons.

Ostium Max Cover, OstiumMAX Cover S should be used with special caution in patients with:

- acute or chronic infection at the surgical site. (E.g., osteomyelitis)
- uncontrolled metabolic diseases such as diabetes, osteomalacia, severe renal or liver disease, thyroid disorder
- autoimmune diseases
- radiotherapy
- prolonged corticosteroid therapy
- high consumption of tobacco

Due to the adhesion to bone tissue and to the elasticity of Ostium Max Cover, OstiumMAX Cover S, the use of bone growth material is recommended to create and maintain space for bone regeneration. In case of membrane exposure (e.g., tissue dehiscence) during the healing phase, the reabsorption time can be accelerated.

PREGNANCY/BREASTFEEDING:

There is no data available on the use of the product during pregnancy or breastfeeding. For safety reasons: Pregnant or nursing women should not receive treatment with Ostium Max Cover, OstiumMAX Cover S.

SIDE EFFECTS:

ACCIDENTS OR ADVERSE EFFECTS SHOULD NOT OCCUR IF THE INSTRUCTIONS AND PRECAUTIONS DESCRIBED ABOVE ARE FOLLOWED.

Some of the unwanted effects may include:

- infection
- Persistent pain

STORE IN A DRY PLACE AT ROOM TEMPERATURE 59-86°F (15-30°C)

Sterilized by gamma radiation.

Condition of sale: Exclusive sale to professionals and health institutions

Technical Director: Pharmacist, Lattini Natacha Verónica, License No. 3838

Authorized by ANMAT (Administración Nacional de Medicamentos, Alimentos y Tecnología Médica [National Administration of Drugs, Foods and Medical Devices]) **PM - 2158-3**

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